

More information.

For current prices and detailed information, please visit: www.lady-comp.com



DEVELOPMENT, MANUFACTURING, WORLDWIDE DISTRIBUTION

 VE Valley Electronics GmbH
Breite 2, 82418
Murnau/Germany
Tel +49 (0) 8841 48928-0
Fax +49 (0) 8841 48928-24
info@lady-comp.de
www.lady-comp.de
info@valley-electronics.com
www.valley-electronics.com

USA SALES AND SERVICE

Valley Electronics LLC
114A West Water Street
Centreville, MD 21617
Tel 1.877.925.LADY(5239)
Fax 1.877.925.5239
info@lady-comp.com
www.lady-comp.com



***Prevent pregnancy naturally.
Plan your family with insight.***
Lady-Comp – the intelligent fertility computer.



LADY-COMP®

Lady-Comp.

Knowing your unique fertility rhythm puts you in control.

The program.

A woman can only become pregnant on six days during each menstrual cycle: five days before ovulation and the day of ovulation. Based on your morning temperature, taken orally, Lady-Comp determines your fertile days and shows when you are ovulating by recording, analyzing and storing your menstrual cycle data. It determines your daily status of fertility and will alert you on the days when intercourse may lead to pregnancy.

It's so easy.

- 1 Take your temperature under your tongue every morning.**
- 2 Enter menstruation on applicable days.**
- 3 Lady-Comp evaluates your data and calculates your fertility status for the next 24 hours:**

● = Infertile

● = Learning phase

● = Fertile

☀ = Ovulation

Reliability is most important.

Lady-Comp's daily fertility display has a Pearl-Index of 0.7, meaning it is 99.3% accurate. The fertility computer is the uncontested leader in terms of accuracy, ease of use and safety.

Upgrades.

You can easily upgrade your Lady-Comp with the baby planning software, Lady-Comp baby, at any time.

Maximize your chances of getting pregnant.

The advanced planning software gives you the insight to plan your family with confidence, from forecasting your most fertile days to providing you with an accurate expected delivery date. Lady-Comp baby can also help those having trouble conceiving by indicating hormonal imbalances and much more.

Fertility calendar and statistics.


The fertility calendar and cycle statistics provide you with valuable information to plan or prevent pregnancy effectively.

- Past and future fertility status.
- Menstruation forecast for the next months.
- Cycle length and length of luteal phase.
- Temperature curve and temperature rise after ovulation.
- Ovulation fluctuation in days.
- Ovulation prognosis and gender prediction*.
- Shows hormonal imbalances*.
- Integrated pregnancy test.
- Shows conception date and birth date*.
- View your cycle details on your Lady-Comp or PC.



*Only for Lady-Comp baby

Medical basics.

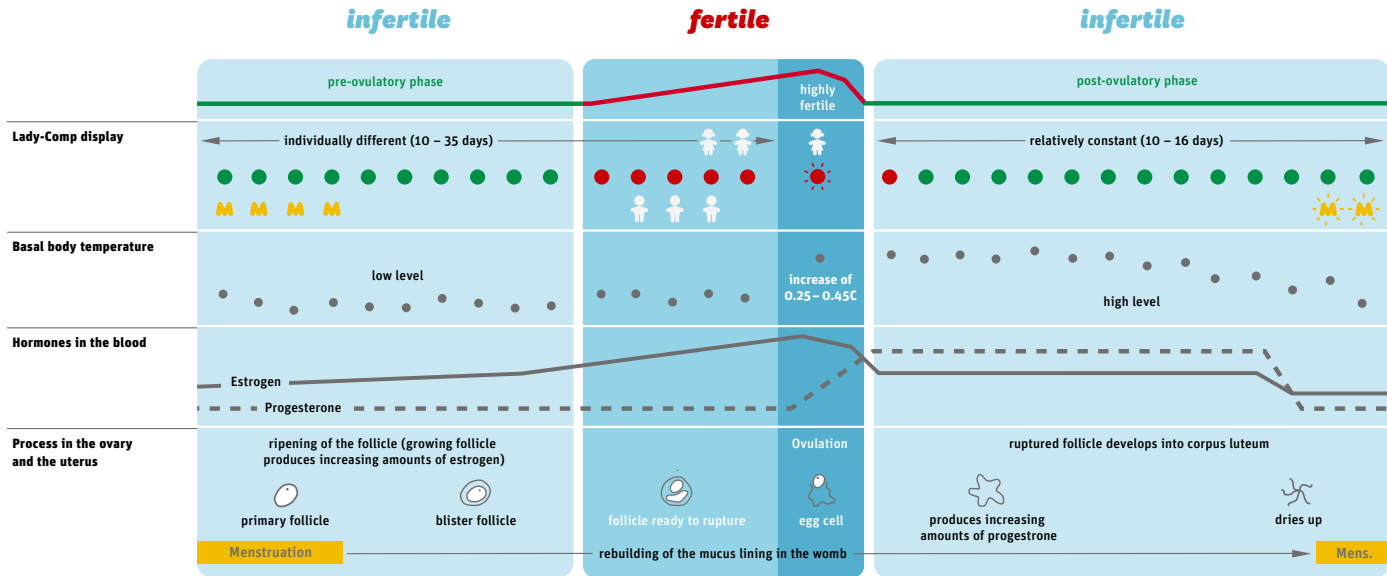

Biological facts:

- Every woman ovulates only once each cycle.
- If on the rare occasion two or more ovulations occur, they will occur within 24 hours.
- After ovulation, the egg can be fertilized for a maximum of 18 hours.
- Sperm can remain active and fertile in the woman's body for a maximum of 5 days.

That means:

On just six days of each cycle can pregnancy occur, five days before ovulation and on the day of ovulation.

Lady-Comp knows:

Lady-Comp uses these biological facts and your individual cycle to reliably indicate when you are fertile or not.